



Extralink Smart Life K-4S Arm Blood Pressure Monitor, White



link to the product:

<https://www.batna24.com/gb/p/blood-pressure-monitors/extralink-smart-life-k-4s-arm-blood-pressure-monitor-white>

Price: **89.00 zł** gross / 72.36 zł net

Manufacturer: EXTRALINK



Referention number: DNXGGMIPJRMMLP

Full product description

Simultaneous measurement of systolic blood pressure, diastolic blood pressure and pulse rate

The device allows simultaneous and accurate measurement of three key health parameters:



The device can simultaneously and accurately measure three key health parameters:



Systolic blood pressure



Diastolic blood pressure



Pulse

Measurement memory

The device is equipped with a memory to **record and store the last 99 results of** systolic blood pressure, diastolic blood pressure and pulse **measurements**. The ability to **accurately track changes** in health parameters over time can be particularly useful in **monitoring the progress of treatment** or for regular health checks. This feature also allows results to be easily shared with professional health care providers during medical visits.

The internal memory of the blood pressure monitor can be used to store results. With dual storage, each measurement result can be stored twice – once as the current result and once as the last result before it. This will store the last 240 measurements in the device's memory.




Simple one-button operation

Intuitive one-button operation makes the device easy to use even for those unfamiliar with medical technology. **Energy-saving function** by automatically turning off the power after three minutes of inactivity, which promotes long-term use of the device.

How to use:

1. Wrap the cuff around the patient's left arm.
2. Making sure that the fingers fit between the cuff and the arm to ensure a proper fit, but not too tightly.
3. The cuff should be applied evenly and smoothly, with the bottom edge of the cuff approximately 2 fingers above the bend of the elbow.
4. Raise your left arm with the cuff upwards, holding the monitor at the same height as your heart. This position will help ensure a more accurate blood pressure measurement.
5. Once you have adjusted the cuff and the patient's arm, press the Start-Stop button on the blood pressure monitor to start the measurement.
6. After reading the result, turn off the blood pressure monitor and remove the cuff from the patient's arm.

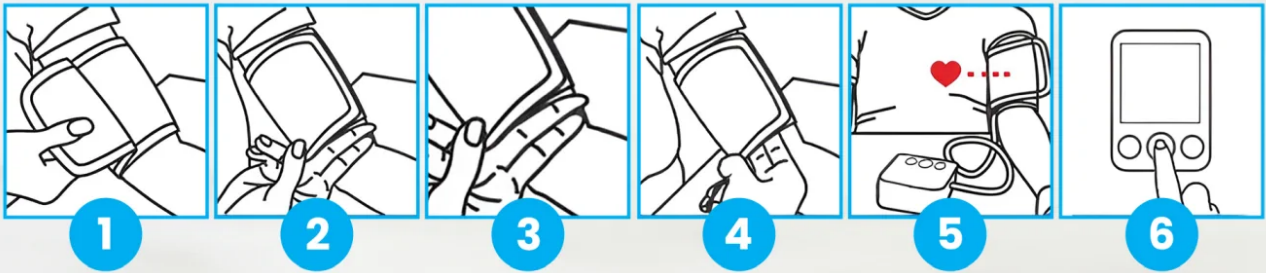

Compact design

The **compact design** and light weight make the device **easy to carry**, allowing you to monitor your health anywhere.



Ideal for home use

The compact design and light weight make the device easy to carry, allowing you to monitor your health anywhere



Upper arm blood pressure monitor

Simultaneous measurement
of systolic blood pressure,
diastolic blood pressure and pulse



Accurate measurement



Storage of the
last 240 results




LED display



One-button operation



Automatic
power switch-off

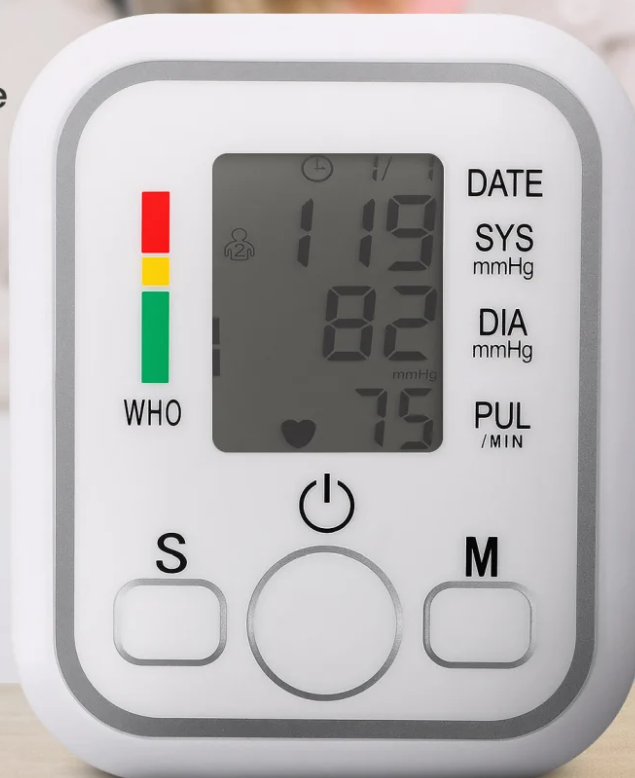

Simple one-button operation



Intuitive one-button operation makes the device easy to use, even for those unfamiliar with medical technology.



An energy-saving function that automatically switches off the power after three minutes of inactivity, promoting long-term use of the device.



1 hour before measurement

