Extralink Smart Life K-69 Kol Damar Tansiyon Aleti, Siyah

5 906168 632099 >

link to the product:

<https://www.batna24.com/tr/p/kan-basinci-monitorleri/extralink-smart-life-k-69-kol-damar-tansiyon-aleti-siyah>

Price: **89,00 zł** gross / 72,36 zł net

Manufacturer: EXTRALINK

Referention number: DNXGGMIPRMMKQ

Full product description

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The device can simultaneously and accurately measure three key health parameters:

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The internal memory of the blood pressure monitor can be used to store results. With dual storage, each measurement result can be stored twice – once as the current result and once as the last result before it. This will store the last 240 measurements in the device's memory.

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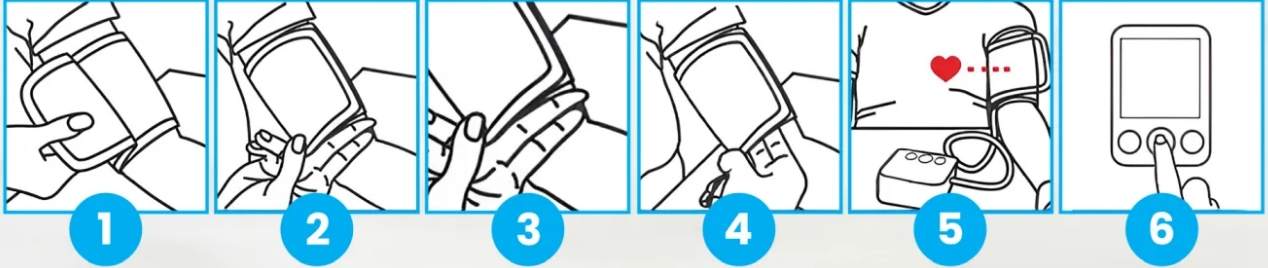
How to use:

1. Wrap the cuff around the patient's left arm.
2. Making sure that the fingers fit between the cuff and the arm to ensure a proper fit, but not too tightly.
3. The cuff should be applied evenly and smoothly, with the bottom edge of the cuff approximately 2 fingers above the bend of the elbow.
4. Raise your left arm with the cuff upwards, holding the monitor at the same height as your heart. This position will help ensure a more accurate blood pressure measurement.
5. Once you have adjusted the cuff and the patient's arm, press the Start-Stop button on the blood pressure monitor to start the measurement.
6. After reading the result, turn off the blood pressure monitor and remove the cuff from the patient's arm.

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Ideal for home use

The compact design and light weight make the device easy to carry, allowing you to monitor your health anywhere

Upper arm blood pressure monitor

Simultaneous measurement of systolic blood pressure, diastolic blood pressure and pulse

Accurate measurement

Storage of the last 240 results

LED display

One-button operation

Automatic power switch-off

Simple one-button operation

Intuitive one-button operation makes the device easy to use, even for those unfamiliar with medical technology.

An energy-saving function that automatically switches off the power after three minutes of inactivity, promoting long-term use of the device.

1 hour before measurement